# How to use your new Rehband Support - 502

UK

- Function** Soft elastic ankle support, which increases the awareness of the foot and ankle. Can be used in all kinds of activities. Please contact a physiotherapist if you have any questions.
- Indications** Instability of the ankle, e.g. sprained and swollen ankle or rheumatism.
- Material Application** Lycra®, cotton, polyamide.
- 1) Place the mid part of the brace against the Achilles tendon and then cross both ends over the upper part of the foot.
  - 2) The end with soft Velcro should be on top and go from the inside to the outside part of the foot.
  - 3) The other end goes from the inside of the foot, then under the foot, before fastening with the soft Velcro on the outside of the foot.
  - 4) Utilise the elasticity in the material for a good support.

## Important Information

- 1) If you have questions regarding application, please contact your local dealer or the prescriber.
- 2) The product is not flameproof, avoid temperature above 120°C/248°F.
- 3) Minor changes are allowed, providing the changes are made through the prescriber\* or local dealer.
- 4) The function of the product must not be changed or compromised, the product should only be used for its original, intended indication.
- 5) Re: neoprene products, time of use should not exceed 3-4 hours. Re: products in other materials, time of use should not exceed 8 hours, unless otherwise is recommended by prescriber.
- 6) If the product contains latex or rubber and you are allergic to latex/ rubber, we advise you to consult a prescriber before using the support. The content of the product is written above.
- 7) If you are allergic to heat, we advise you to use a product that not contains neoprene.

\*Re: Medical- products, the prescriber must be a qualified doctor, PT, OT or Orthopaedic Engineer.

## Washing Instructions

- 1) Close all Velcro.
- 2) Remove detachable splints.
- 3) Mesh laundry bag is recommended.
- 4) Stretch product while wet and let dry.

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